




Lapiena

OUR ETHOS

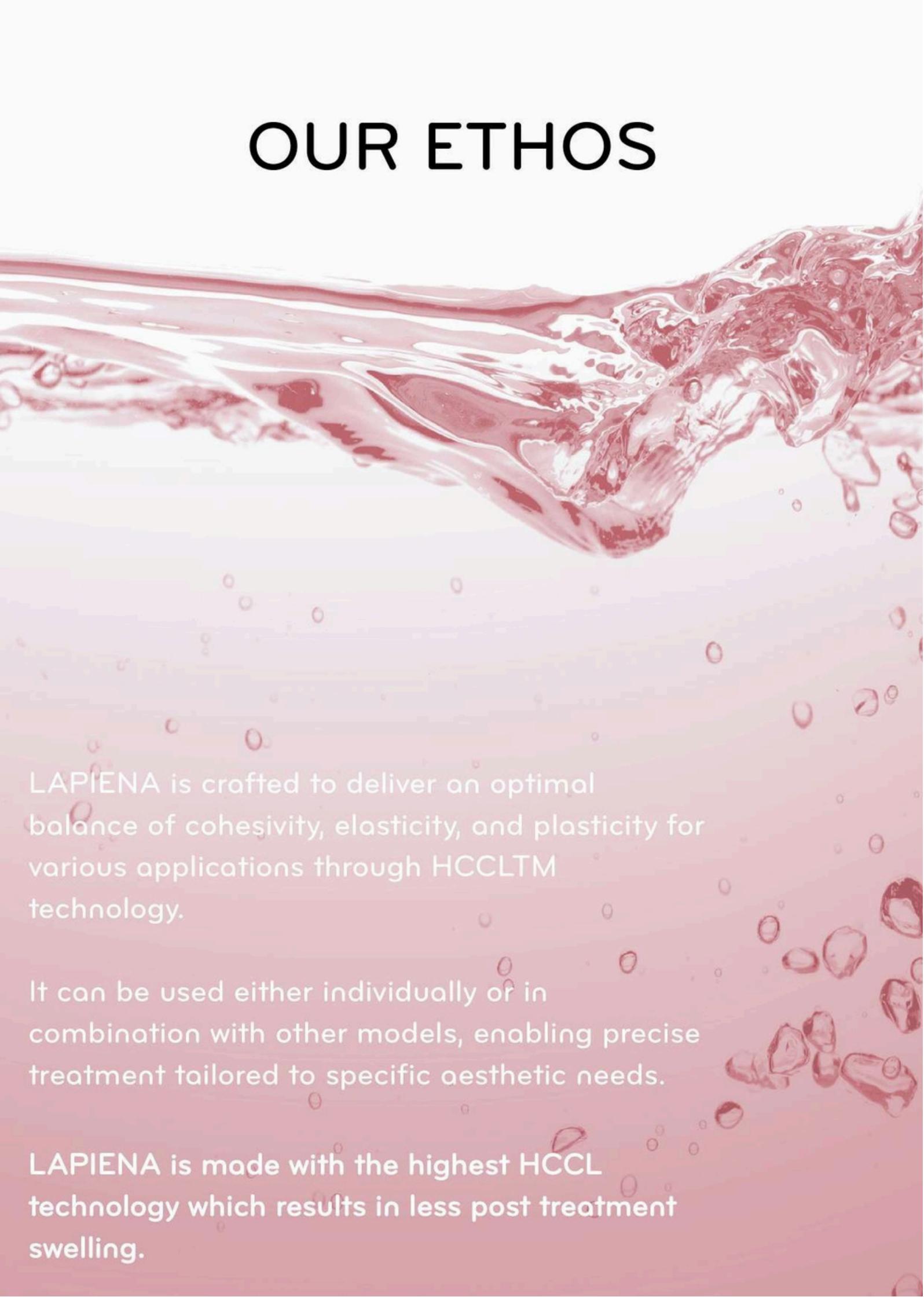

LAPIENA is crafted to deliver an optimal balance of cohesivity, elasticity, and plasticity for various applications through HCCLTM technology.

It can be used either individually or in combination with other models, enabling precise treatment tailored to specific aesthetic needs.

LAPIENA is made with the highest HCCL technology which results in less post treatment swelling.

CLASSIC



SUB-Q



DEEP



01 Clinical Results

Our clinical trial verified that LAPIENA an effective and safe medical device for the correction of nasolabial fold and not inferior to the reference device*. (*The reference device is approved by US FDA)

TEST METHOD -

Wrinkle Severity Rating Scale (WSRS) and Global Aesthetic Improvement Scale (GAIS) were assessed with 68 patients by independent evaluators for 24 weeks after nasolabial fold injection.

02 Safety Assessment

LAPIENA has been evaluated as a safe medical device through clinical tests, which showed no clinically significant abnormalities.

03 Patient Satisfaction

The addition of Lidocaine offers a pain-relieving effect during the procedure, thereby enhancing patient comfort.



01 CE Mark, ISO 13485

Lapiena is CE certified, indicating that the product meets the European Union's health, safety, and environmental standards for Class III medical devices.

02 Approved Ingredient

The Hyaluronic Acid used is of pharmaceutical grade and is approved with FDA DMF and EDQM certifications.

03 HCCL technology

The safety and effectiveness of Lapiena have been demonstrated in a comparative clinical trial.

04 Proved Safety

A reinforced 3D matrix design that ensures maximum safety and achieves ideal rheological characteristics.



INTRODUCING CLASSIC

RECOMMENDED INJECTION AREAS

Crows Feet

Glabella Lines

Perioral Wrinkles


Forehead Lines

Tear Troughs

Earlobe Augmentation

Lip Contouring

Marionette line

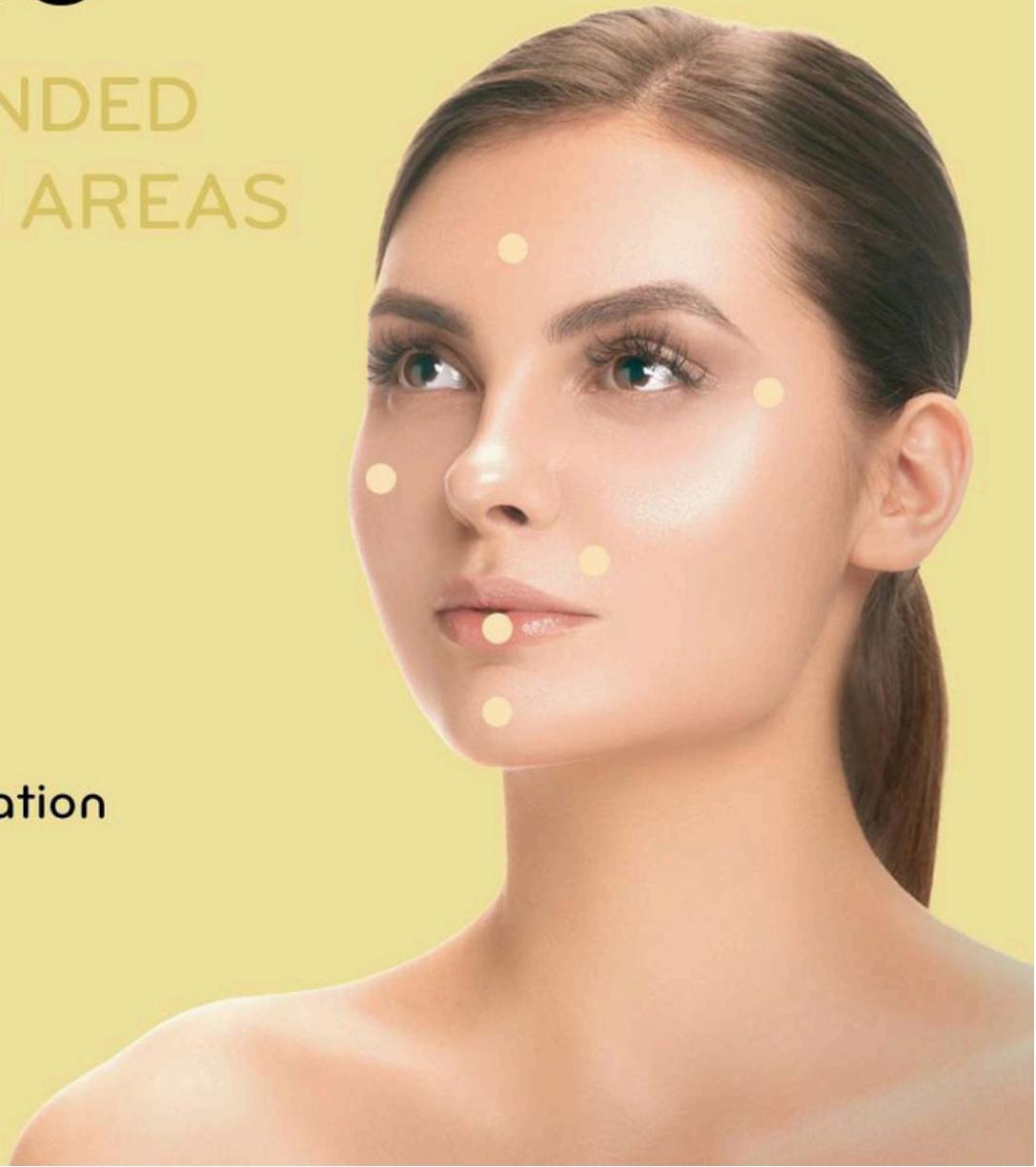



INTRODUCING DEEP

RECOMMENDED INJECTION AREAS


Nasolabial Lines

Marionette Lines

Lip volumizing

Glabella Lines

Forehead Lines

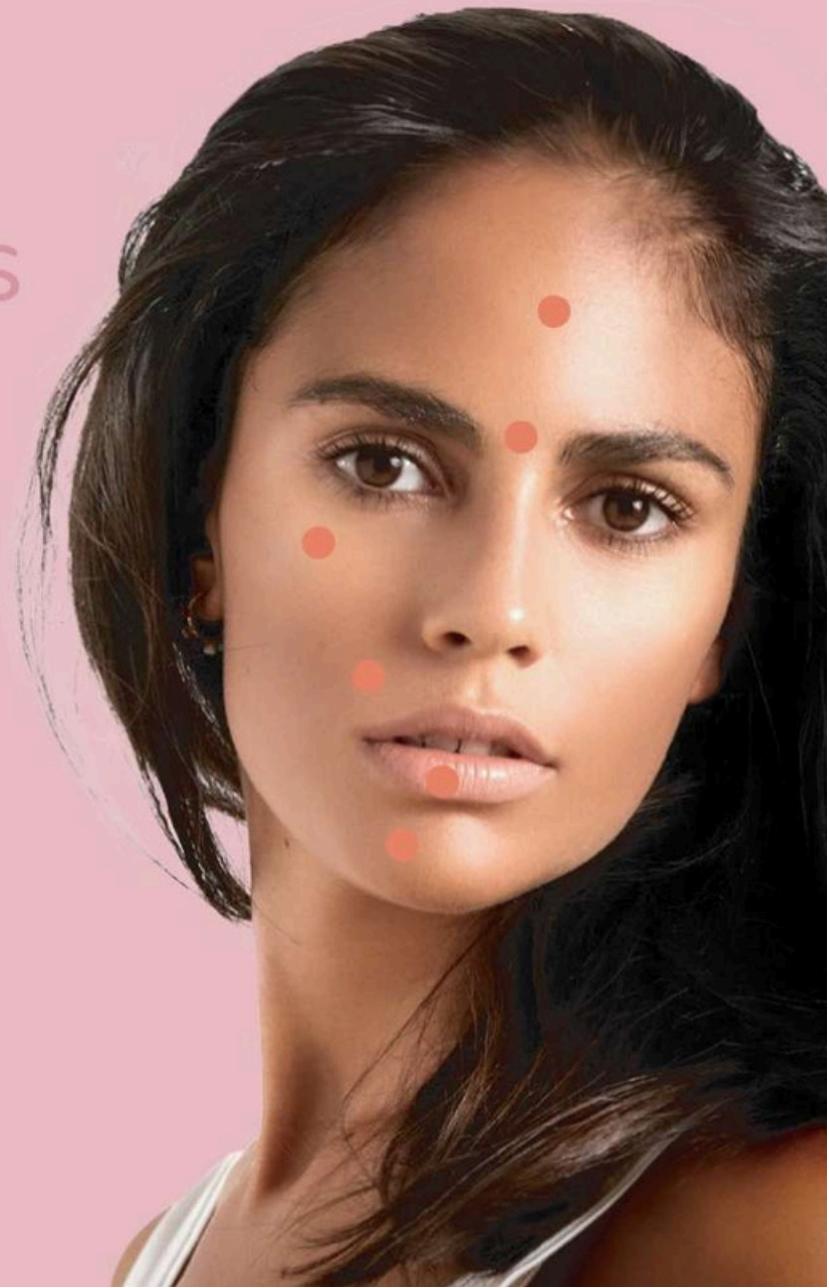





INTRODUCING SUB-Q

RECOMMENDED INJECTION AREAS

- Cheek Augmentation
- Nose Augmentation
- Jawline Augmentation
- Forehead Augmentation

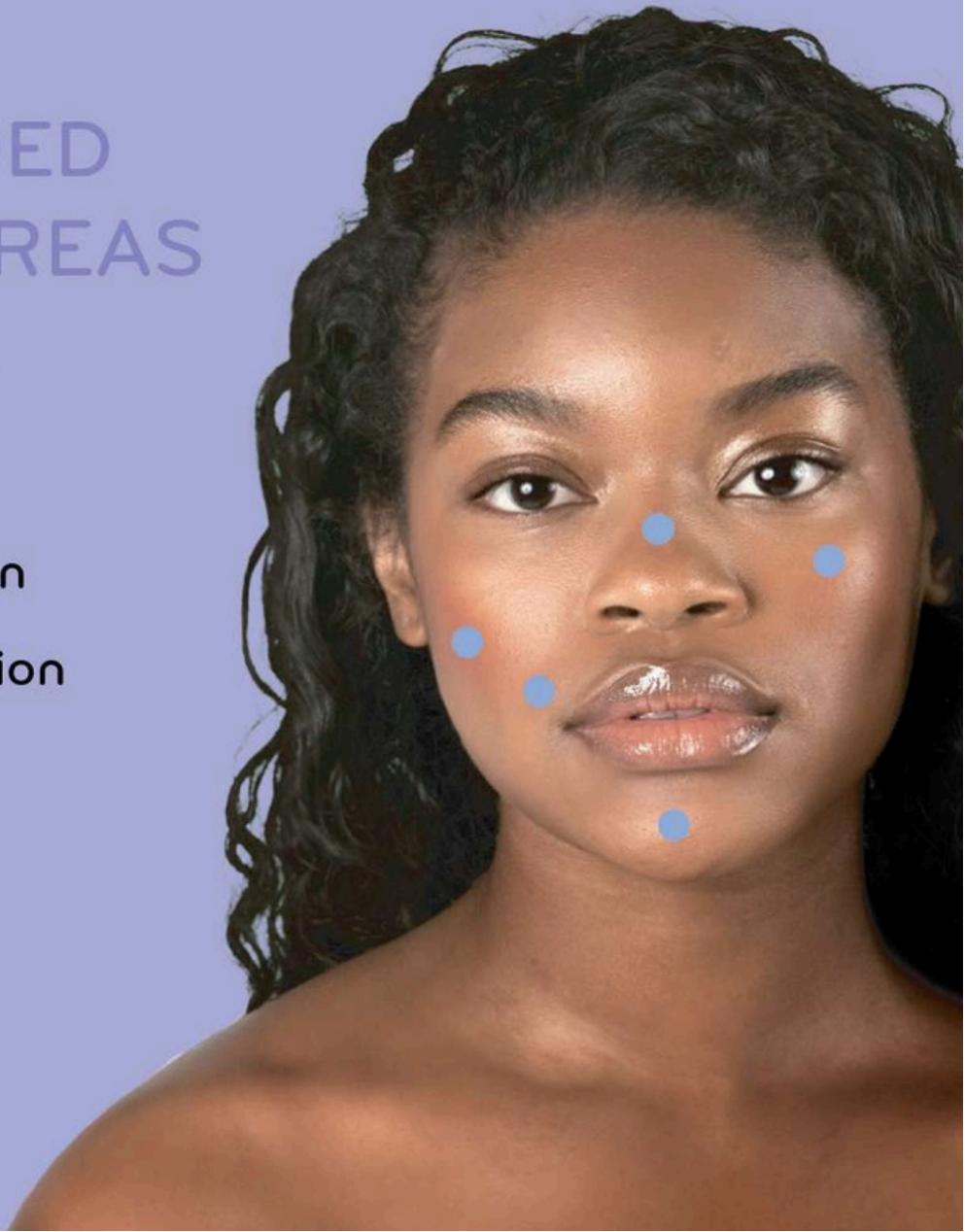


Classic

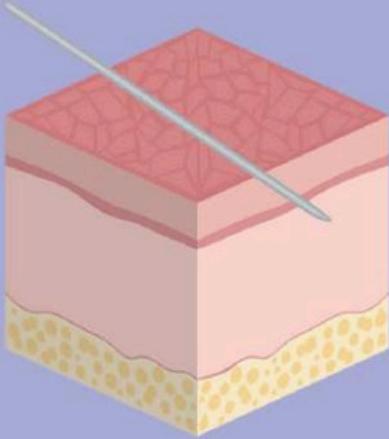

Into the upper dermis at approximately 15 degrees.

LAPIENA CLASSIC is the softest and thinnest option in the LAPIENA range, specifically designed for correcting fine to moderate lines.


Sub-Q

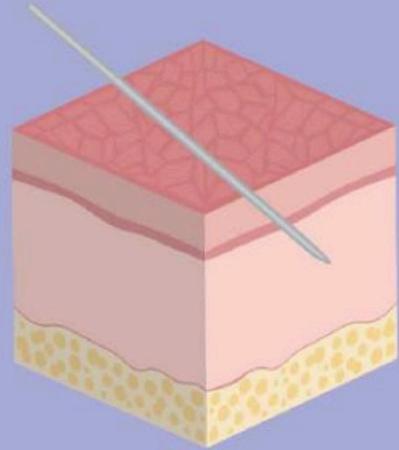

Into the deep dermis and periosteum at approximately 30~45 degrees.

LAPIENA Sub-Q is the thickest formulation within the LAPIENA range, specifically designed for restoring facial shape and reconstructing volume loss.

Deep

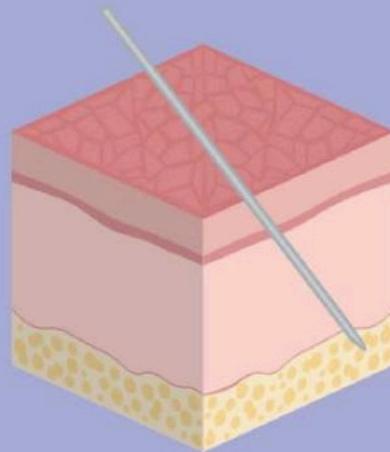

Into the medium and deep dermis at approximately 30 degrees.

LAPIENA DEEP offers enhanced viscoelastic properties, making it ideal for addressing deep folds and restoring natural volume in facial areas subjected to frequent and repetitive movements.

Classic**Deep****Sub-Q**

Composition	Cross-linked HA Lidocaine 0.3%	Cross-linked HA Lidocaine 0.3%	Cross-linked HA Lidocaine 0.3%
HA Content	20mg/ml		
Volume	1ml x 1 Syringe		
Needle	25G 13mm	27G 13mm	30G 13mm
Fine wrinkles	Frontal Cheek, Nasolabial folds, Lips, Under Eyes, Cheekbone, Fine Wrinkles	Lips, Forehead, Temple, Deep Wrinkles, Nasolabial Folds, Glabella	Nose bridge, Nose Tip, Forehead, Laugh Lines, Cheek, Chin
Packing	1 x 0.1ml syringe 1 x 27G needles 1 x 30G needles	1 x 1.1ml syringe 1 x 25G needles 1 x 27G needles	1 x 1.1ml syringe 1 x 25G needles 1 x 27G needles

Cohesivity - The ability of a gel to stick together and thereby maintaining gel integrity.

Elasticity - The ability of a gel to return to its initial shape after being subjected to strain.

Plasticity - The ability of a gel to be shaped and modelled.

Classic

Cohesivity	✿ ✿ ✿
Elasticity	✿ ✿ ✿
Plasticity	✿ ✿ ✿


Sub-Q

Cohesivity	✿ ✿ ✿
Elasticity	✿ ✿ ✿
Plasticity	✿ ✿ ✿

Deep

Cohesivity	✿ ✿ ✿
Elasticity	✿ ✿ ✿
Plasticity	✿ ✿ ✿

THE BENEFITS OF USING **Lapiena***

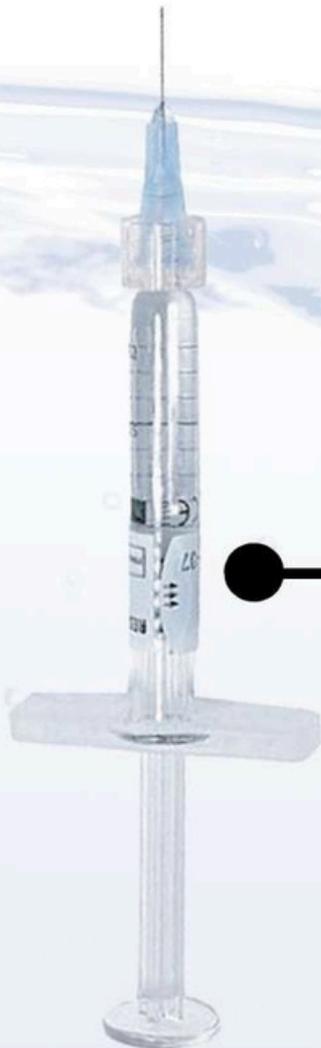

- Immediate results with little or no downtime.
- Decreases common signs of aging.
- Restores volume
- Non-invasive
- Long lasting

DISTRIBUTION OPPORTUNITY

We are a UK based company that can distribute products safely and efficiently to anywhere across the country.



Lapiena

www.lpgclinicswholesale
info@lpgclinicswholesale.com

LPG
CLINICS WHOLESALÉ